


MELAflash

CF-Card-Printer

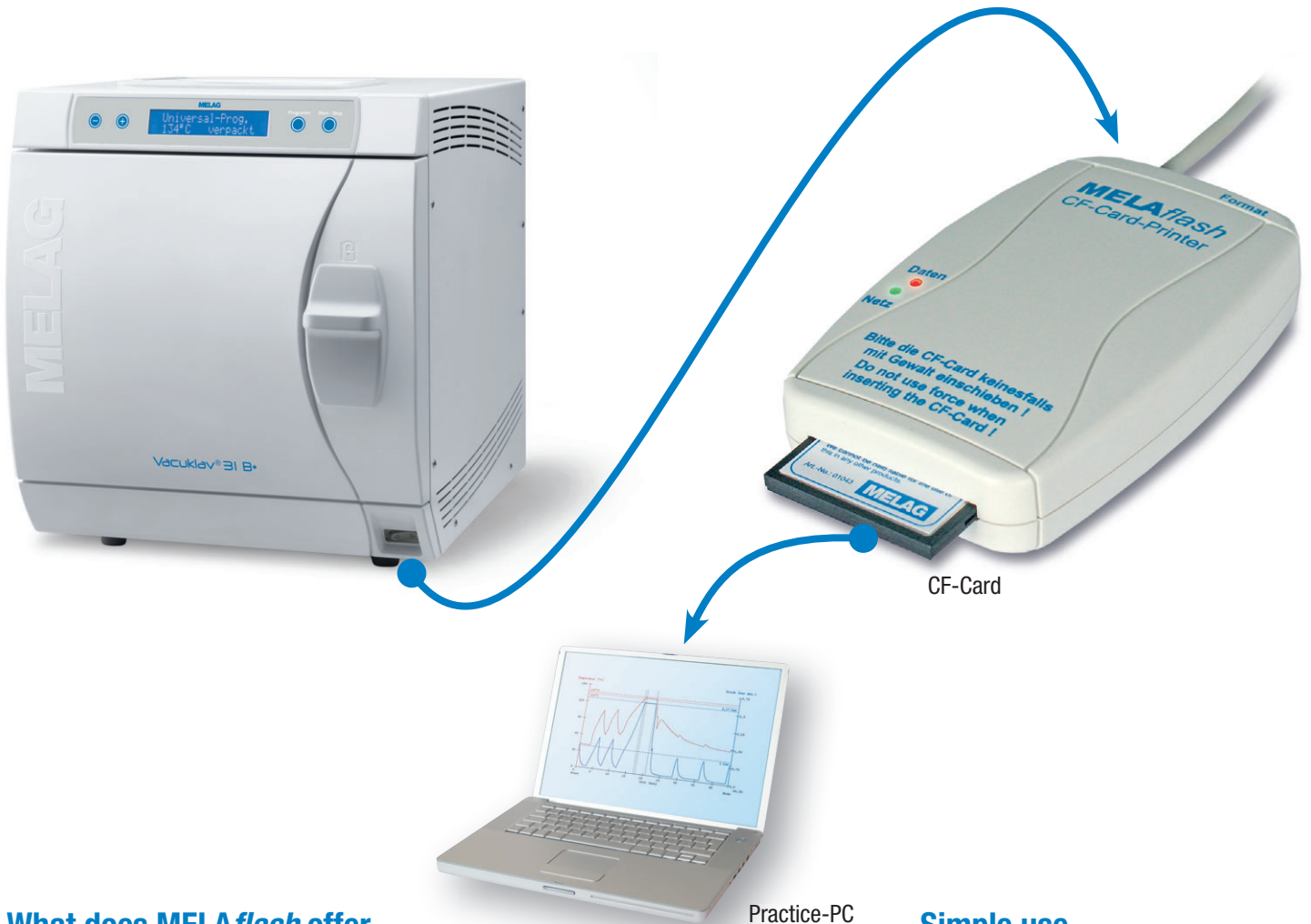

What does MELAflash offer

The MELAflash system consists of the CF-Card-Printer, the MELAflash CF-Card and the MELAflash Card-Reader.

These three components are used to make the documentation for the well-tried MELAG autoclaves of the series Pro-Class, S-Class, Cliniklav®25 and MELAtronic®EN definitely more simplified and comfortable.

Without any further connection of the autoclave to the practice network or an external printer, the sterilization records can be very simply saved on a PC. This way you satisfy the requirements of durable reproducibility of the successful sterilization in your practice.

Saving and displaying the data

After transmitting the data over the supplied MELAflash Card-Reader to the practice PC, the records can be examined with any word processing program or any spreadsheet, supplemented with comments or printed out on a printer connected to the PC.

More comfortable is the documentation with the software MELAview, developed by MELAG.

Simple use

The MELAG autoclave is set to „External PC“ on the setup menu. If the MELAflash CF-Card-Printer is connected to the serial interface of the autoclave, the sterilization data are automatically saved on the MELAflash CF-Card.

Scope of delivery / Technical data

The MELAflash CF-Card-Printer includes the MELAflash CF-Card as well as the MELAflash Card-Reader.

Dimensions:

Width: 7.5 cm x depth: 11.5 cm x height: 2.5 cm.

Quality - made in Germany

MELAG

www.melag.com